TrueLok™ TL-HEX

Ring Fixation System

TrueLok Hexapod System®

EN	TrueLok and TL-HEX Dynamization Module (Dynamo) and Radiolucent Dynamizer 2	ZH	TrueLok 和 TL-HEX 动力化模块 (Dynamo) 以及透视型动力计 67
IT	Modulo di dinamizzazione TL-HEX e TrueLok (Dynamo) e dinamizzatore radiotrasparente 7	CS	Dynamizační modul (Dynamo) a radiopropustný dynamizér pro systémy TrueLok a TL-HEX 72
FR	Module de dynamisation TL-HEX et TrueLok (Dynamo) et dynamiseur radiotransparent 12	PL	Moduł dynamizacyjny TrueLok i TL-HEX (Dynamo) i dynamizator przezierny dla promieni RTG 77
DE	TrueLok- und TL-HEX Dynamisierungsmodul (Dynamo) und strahlendurchlässiger Dynamizer 17	SL	Dinamizacijska modula TrueLok in TL-HEX (Dynamo) ter radiolucentni dinamizer 82
ES	Módulo de Dinamización TrueLok y TL-HEX (Dynamo) y Dinamizador Radiotransparente 22	KO	TrueLok 및 TL-HEX 역동화 모듈 (다이내모) 과 방사선 투과성 다이내마이저 87
DA	TrueLok- og TL-HEX-dynamiseringsmodul (Dynamo) og gennemlysningsbare dynamisator 27	AR	وحدة التفعيل الديناميكي TrueLok و TL-HEX (دينامو) والمفعل الديناميكي الإشعاعي 92
FI	TrueLok- ja TL-HEX-dynamisointimoduuli (Dynamo) ja radiolusentti dynamisoija 32	BG	Модул за динамизация TrueLok и TL-HEX (динамо) и радиопрозрачен динамизатор 97
NO	TrueLok og TL-HEX dynamiseringsmodul (dynamo) og røntgengjennomsiktig dynamisator 37	LT	„TrueLok“ ir „TL-HEX“ dinamizavimo modulis („Dynamo“) ir radiacinei spinduliutei pralaidus dinamizavimo įtaisas 102
NL	TrueLok- en TL-HEX-dynamisatiemodule (dynamo) en radiolucente dynamizer 42	MS	TrueLok dan Modul Dinamisasi (Dynamo) TL-HEX dan Dinamizer Radiolusen 107
PT	Módulo de dinamização TrueLok e TL-HEX (Dynamo) e dinamizador radiotransparente 47	RO	Modul de dinamizare TrueLok și TL-HEX (Dinamo) și dinamizator radiotransparent 112
SV	TrueLok- och TL-HEX-dynamiseringsmodul (Dynamo) och radiotransparent dynamiserare 52	TR	TrueLok ve TL-HEX Dinamizasyon Modülü (Dinamo) ve Radyolusent Dinamize Edici 117
EL	Μονάδα δυναμοποίησης TrueLok και TL-HEX (Dynamo) και ακτινοδιαπερατή συσκευή σδυναμοποίησης 57	HR	TrueLok i TL-HEX dinamizacijski modul (Dynamo) i radiolucentni dinamizator 122
JA	TrueLok および TL-HEX ダイナミゼーションモジュール (Dynamo) ならびにラジオルーセントダイナマイザー 62	LV	TrueLok un TL-HEX dinamizācijas modulis (Dynamo) un rentgencaurlaidīgs dinamizators 127

ORTHOFIX Srl
Via Delle Nazioni 9 - 37012 Bussolengo (Verona) - Italy
Telephone +39 045 6719000 - Fax +39 045 6719380

PQ.TLD.D 11/23 (0425655)

EN

Instructions For Use (IFU) are subject to change; the most current version of each Instruction For Use is always available online.

Important information - please read prior to use

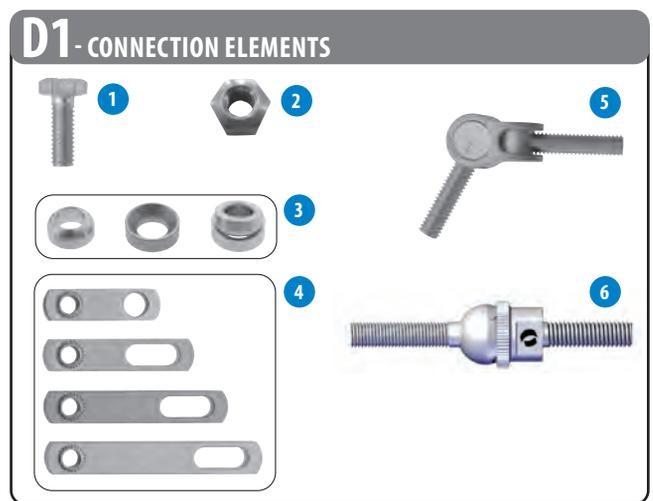
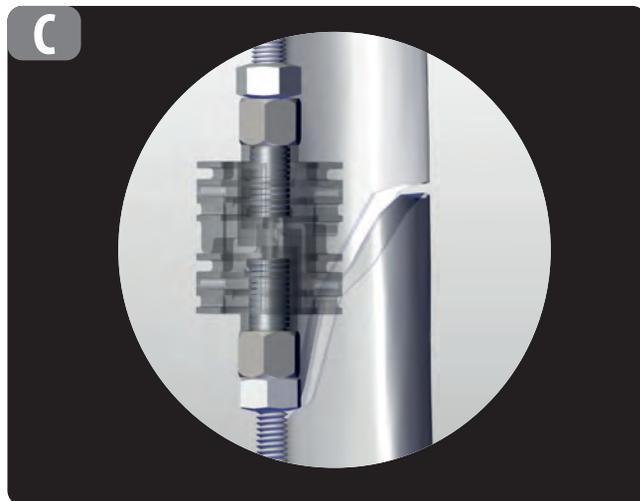
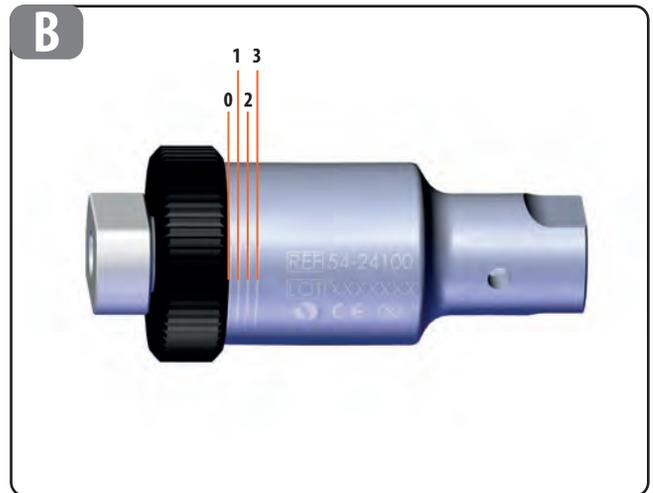
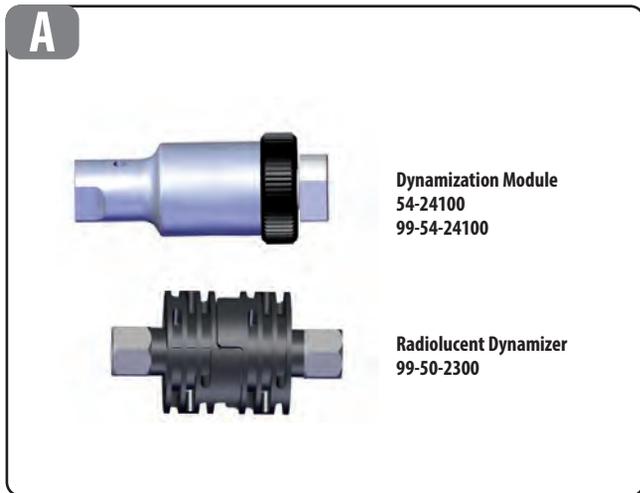
See also instruction leaflet PQTLK for the TrueLok™ Systems

TrueLok™ TL-HEX
 Ring Fixation System TrueLok Hexapod System*

TrueLok and TL-HEX Dynamization Module (Dynamo) and Radiolucent Dynamizer

Manufacturer name
 ORTHOFIX SRL, via Delle Nazioni 9, 37012 Bussolengo (VR) Italy
 Tel. 0039 (0) 45 6719000 - Fax 0039 (0) 45 6719380





D2 - CONNECTION ELEMENTS




Part #	Description
54-1050 54-1050M 54-1010 54-1010M 54-1018	1 BOLTS
54-1008 54-1008M	2 NUTS
54-1150 54-1150M	3 CONICAL WASHER
55-10340 55-11670 55-11671 55-11680	4 PLATES
55-1176	5 UNIVERSAL HINGE
99-50-2310M	6 SPHERICAL JOINT

E - INSTRUMENTS

Part #	Qty
54-1154	2
52-1018	1

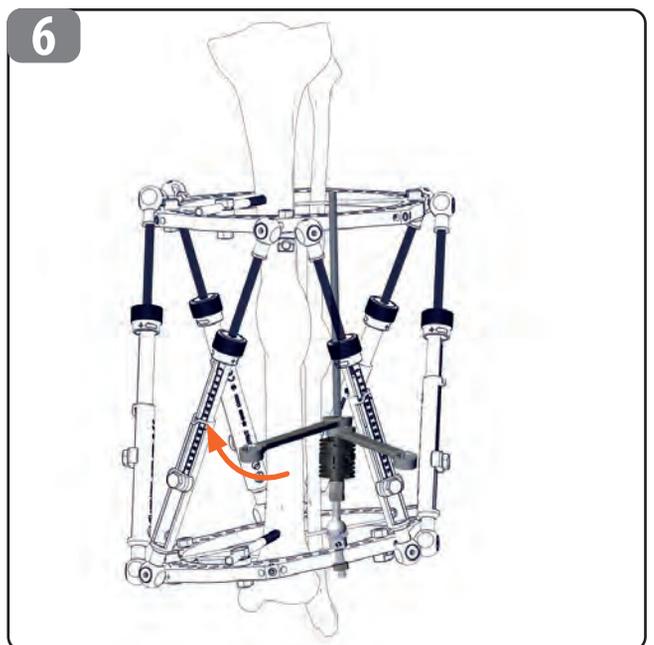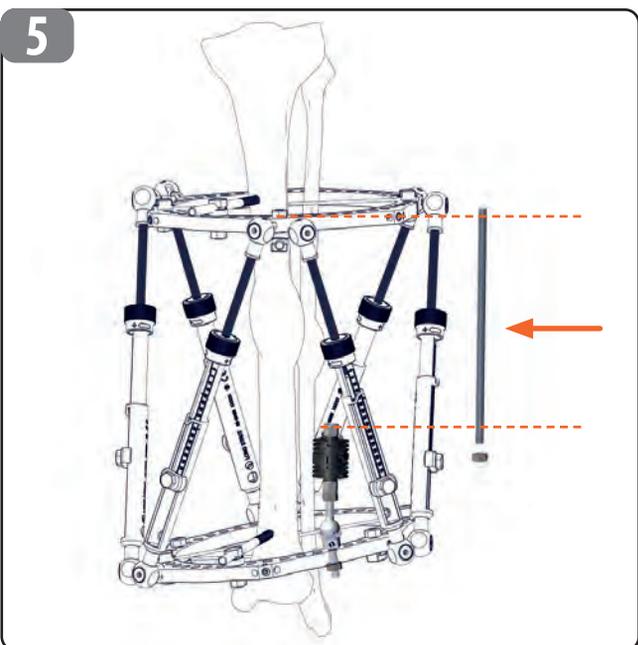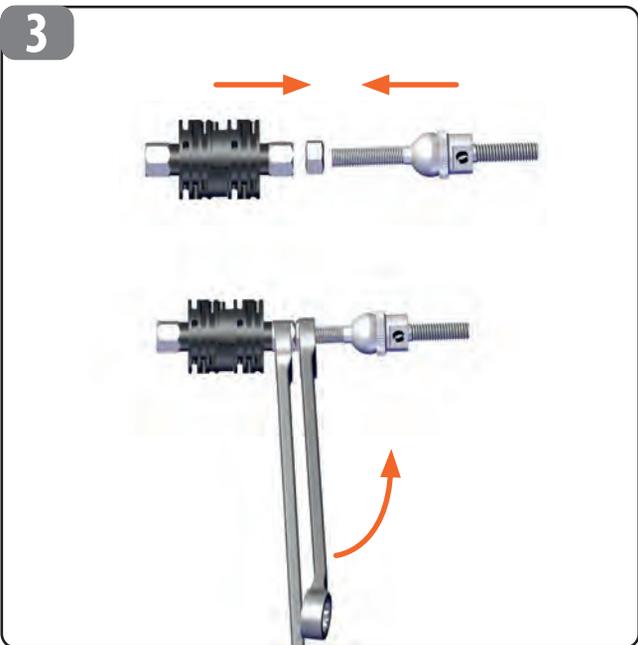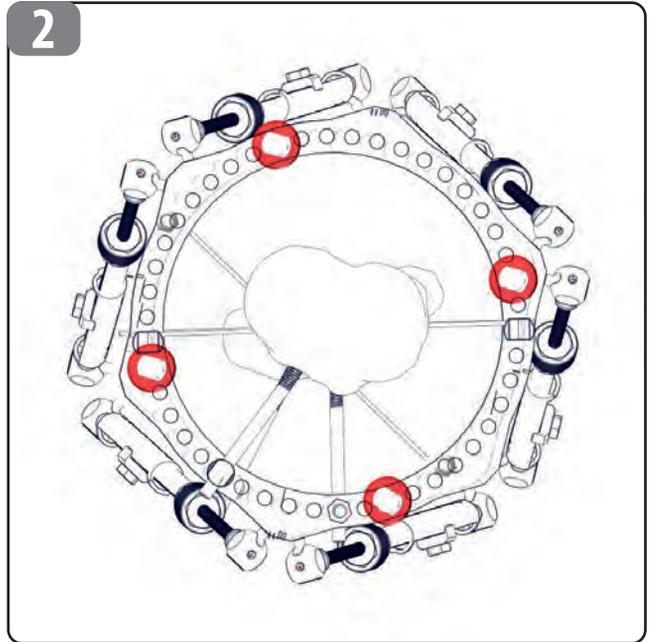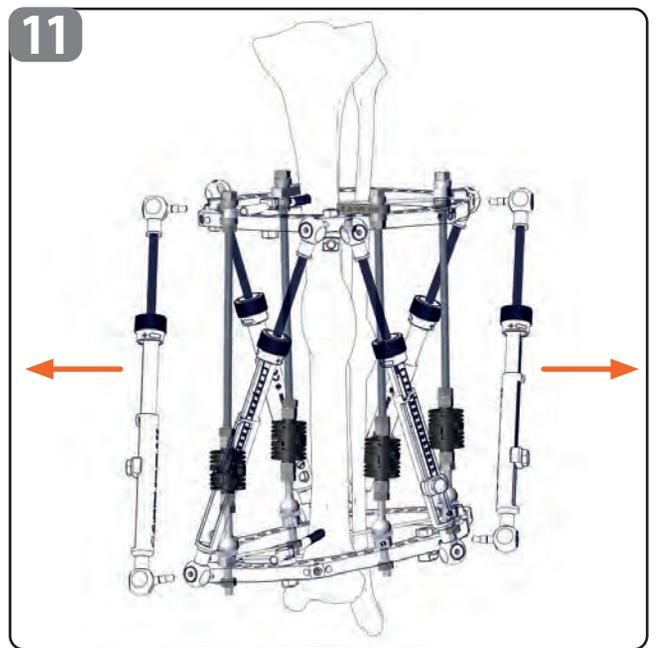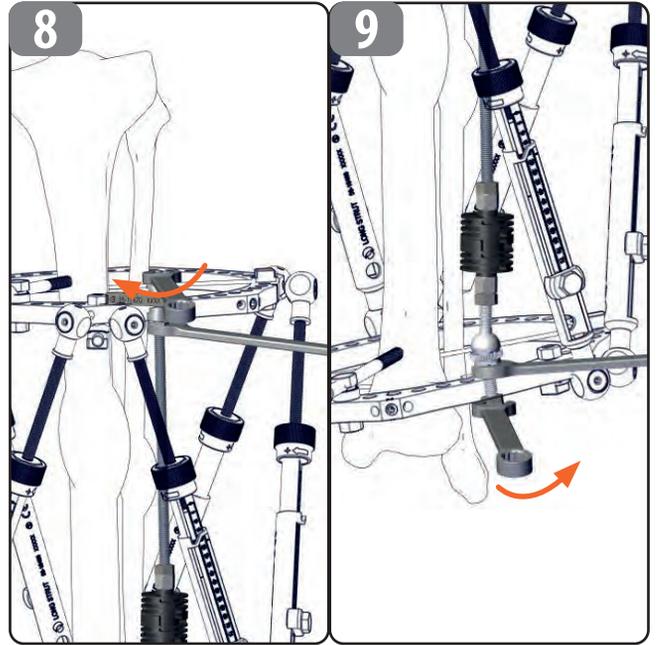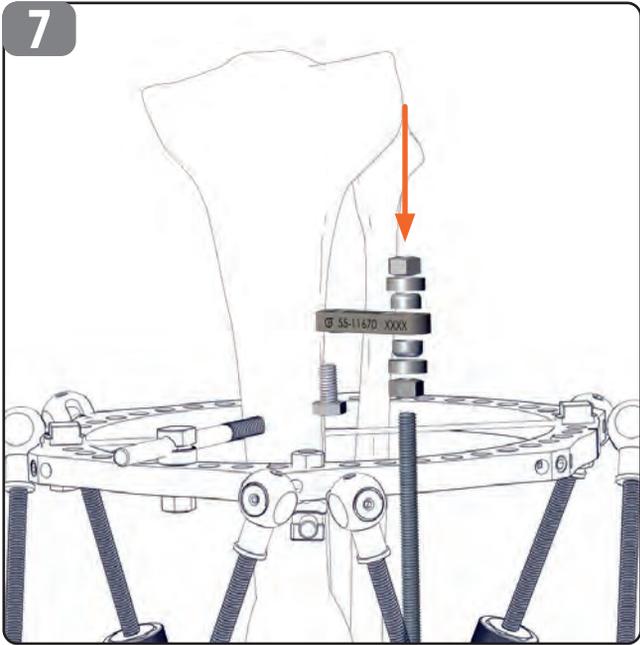TrueLok is a trademark or registered trademark exclusively licensed to Orthofix Medical Inc., and its affiliate companies. All rights reserved.

TrueLok and TL-HEX Dynamization Module and Radiolucent Dynamizer

DESCRIPTION

The Dynamization Module and the Radiolucent Dynamizer (see image A) are compatible with TrueLok (TL), TrueLok HEX (TL-HEX) and TrueLok EVO (TL-EVO) systems. They can be used to dynamize an existing frame towards the end of the treatment or at any stage where dynamization of the fracture callus or regenerate is required.

Both dynamization modules previously described will be referred to as dynamizers.

IMPORTANT INFORMATION

For important medical information, maintenance and sterilization consult Instruction for Use PQTLK or PQEVO.

It is fundamental for the correct application and performance of the dynamizers to set them up at the same level and as parallel as possible to the bone axis.

Two choices of dynamizers are available and it is up to the Healthcare Professionals (HCP) to select the appropriate tool for the treatment of the specific patient.

The main differences between the two dynamizers are the following:

- The metallic radiopaque Dynamization Module allows a controlled micromotion amplitude up to 3mm by setting it through the reference lines, from 0 to 3mm of dynamization, (see image B).
- The Radiolucent Dynamizer allows a controlled micromotion amplitude by 3mm. This is considered to be the less rigid version of its metallic counterpart.

APPLICATION OF DYNAMIZERS

1. Before positioning the dynamizer it is recommended to check which one of the two rings of the frame is more perpendicular to the bone. The reference image (frame of two rings with TL-HEX struts) is only representative of a possible frame.
2. Four dynamizers must be used. Consider that they must be placed on the same ring, as equidistant as possible from each other. Dynamization occurs on the axis of dynamizer.
3. Screw into the first dynamizer one end of the spherical joint with a nut and fix it with two wrenches: hold in fixed position the wrench attached to the dynamizer and rotate the one attached to the nut.
4. Fix the spherical joint with the dynamizer on the ring that is less perpendicular to the bone, using a nut. Tighten by hand.
5. Choose one of the threaded rods available in different lengths based on the distance between the free-end of the dynamizer and the second ring.
6. Screw into the free-end of the dynamizer the threaded rod chosen previously with a nut; tighten first by hand and then, utilizing two wrenches, hold in fixed position the wrench attached to the dynamizer and rotate the wrench attached to the nut.
7. Attach the free-end of the threaded rod to the second ring using the following: two set of conical washers, nuts and, if necessary, the suitable plate. Tighten then by hand.
8. Adjust the dynamizer orientation to make it as parallel as possible to the bone axis. Depending on the relative positions of the rings other connection elements might be necessary: refer to picture D1 and D2. This step is fundamental for the correct application and performance of the dynamizers. Once the correct position has been reached, tighten the dynamizer system to the ring more perpendicular using two wrenches rotating the wrench attached to the nut external to the ring.
9. Tighten the other end of the dynamizer system using provided wrenches, rotating the one attached to the nut external to the ring.
10. Apply the other three dynamizers as described above (steps 3 to 9).
11. Once all the dynamizers are positioned, proceed with the removal of the TL Hex Struts.
12. The final dynamization structure set up is illustrated in picture 12.
13. In case of a TrueLok frame, remove one connector at a time and replace it with a dynamizer structure.

NOTE FOR USE

Do not use Orthofix products in conjunction with those of other manufacturers, unless otherwise specified, as the combination is not covered by the necessary validation.

Depending on the setting, the Dynamization Modules are capable of elastic shortening in a range of 0-3mm when load is applied. Refer to the reference lines (from 0 to 3mm of dynamization) to ensure this. The Dynamization Modules are not capable of lengthening.

WARNING

Do not use in applications that bridge an articulation.

PRECAUTIONS

1. In order to dynamize correctly, four dynamization modules or four radiolucent dynamizers must be placed on the same ring.
2. The direction of dynamization is along the axis of the dynamizers: hence, carefully define their orientation based on the intended direction of dynamization.
3. The same level of dynamization must be set on all dynamization modules (code 55-24100 or 99-54-24100) using the reference lines.
4. All the dynamizers must be fixed as parallel as possible to the bone axis.

Symbol		Description	
		Consult instructions for use or consult electronic instructions for use	Caution: Consult instructions for use for important cautionary information
		Single Use. Do not re-use	Orthofix note: discard appropriately after the use (treatment) on patient
		Sterile. Sterilised by irradiation	
		Do not re-sterilize	
		Non-sterile	
		Double sterile barrier system	
		Catalogue number	Batch code
		Use-by date (year-month-day)	
		CE marking in conformity to applicable European Medical Device Directives/Regulations	
		Date of manufacture	Manufacture
		Do not use if package is damaged and consult instructions for use	
Rx Only		Caution: Federal law (USA) restricts this device to sale by or on the order of a physician	

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TrueLok은 Orthofix Medical Inc. 및 그 계열사에 독점적으로 라이선스가 부여된 상표 또는 등록 상표입니다. All rights reserved.
Orthofix Srl